

IVOCLAR VIVADENT ALL CERAMICS

PREPARING FOR CEMENTATION – GLASS CERAMICS

	IPS Empress Esthetic IPS Empress CAD	IPS e.max Press		IPS e.max CAD		IPS e.max Ceram
Indication	Inlays, onlays, veneers, anterior + posterior crowns	Veneers ²⁾ , partial crowns ²⁾ , anterior + posterior crowns, 3-unit bridges up to the 2 nd premolar		Veneers ²⁾ , partial crowns ²⁾ , anterior + posterior crowns		Veneers ²⁾
Cementation method	adhesive ✓	adhesive ✓	self-adhesive*/ conventional ✓	adhesive ✓	self-adhesive*/ conventional ✓	adhesive ✓
Blasting	—	—		—		—
Etching	60 sec. with IPS Ceramic Etching Gel	20 sec. with IPS Ceramic Etching Gel		20 sec. with IPS Ceramic Etching Gel		20 sec. with IPS Ceramic Etching Gel
Conditioning / Silanization	60 sec. with Monobond-S	60 sec. with Monobond-S	— ¹⁾	60 sec. with Monobond-S	— ¹⁾	60 sec. with Monobond-S
Cementation system	Variolink® Veneer Variolink® II Multilink® Automix	Variolink® Veneer Variolink® II Multilink® Automix	Vivaglass® CEM	Variolink® Veneer Variolink® II Multilink® Automix	Vivaglass® CEM	Variolink® Veneer Variolink® II

¹⁾ For self-adhesive cementation, silanization is necessary.

²⁾ Partial crowns and veneers must be adhesively cemented

* self-adhesive powder-liquid systems



Please read the corresponding Instructions for Use

ivoclar
vivadent®